



ResMed

Hitwin
HEALTHCARE

Respiratory • Sleep Therapy • Cardiology

AirFit™ F30




Simplicity wrapped in high performance

Introducing ResMed AirFit™ F30 – our minimal-contact full face mask that outperforms Phillips Respironics® DreamWear™ full face and Amara View™ in key areas that matter to you – comfort,¹⁻³ fit,^{2,3} seal¹⁻³ and quietness.⁴⁻⁶ Plus, it makes ordering and inventory management more efficient for your business with the fewest SKUs.*

www.hitwinhealthcare.com




AirFit F30 highlights

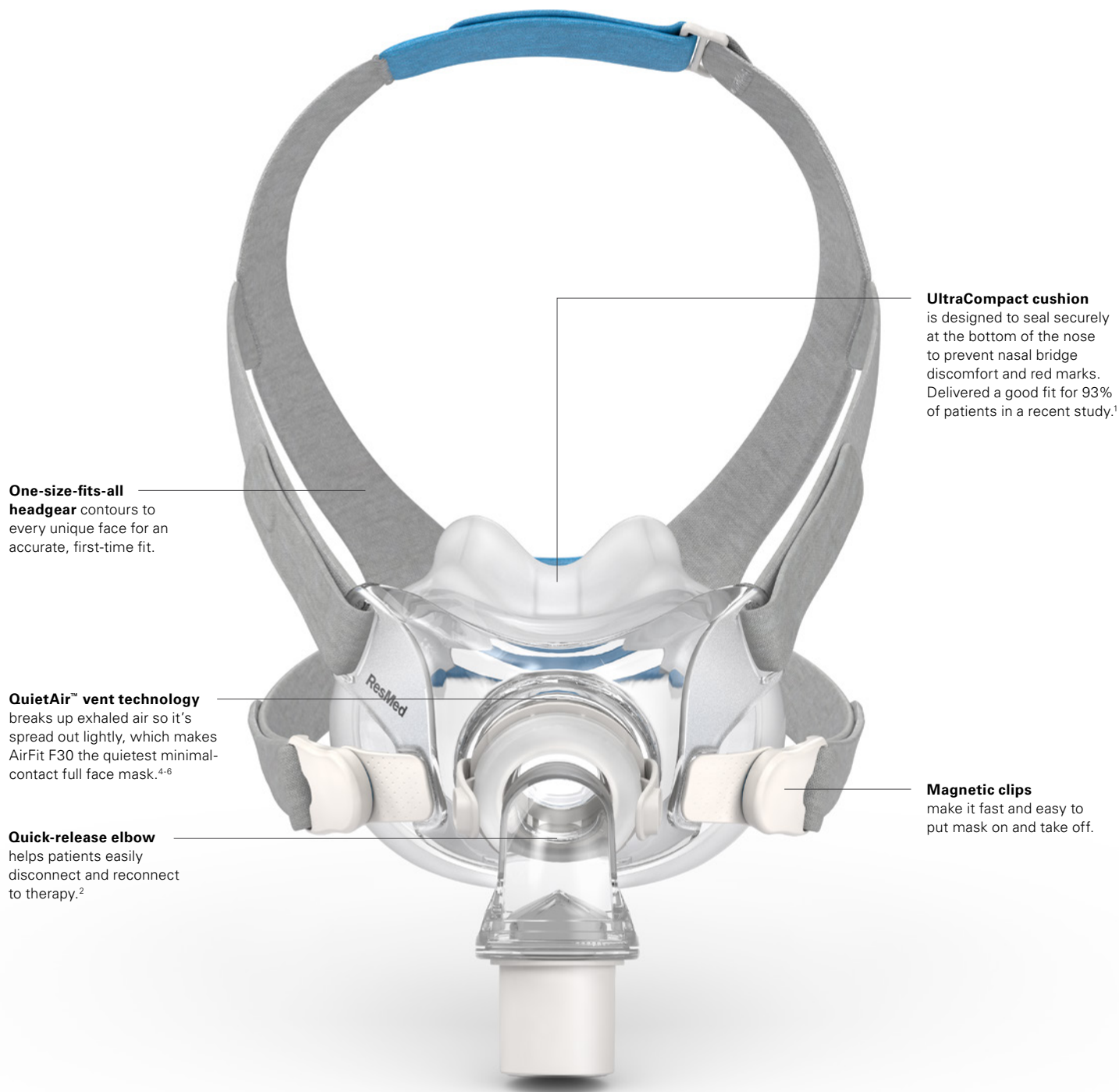

AirFit F30 Product Codes

Complete mask systems (includes frame system + headgear)	Cushions included	Product code
AirFit F30: Small	S	64100
AirFit F30: Medium	M	64101
Frame systems (includes frame, cushion and elbow)		
AirFit F30 Frame System: Small	S	64155
AirFit F30 Frame System: Medium	M	64156

Accessories	Product code
AirFit F30 Cushion: Small	64150
AirFit F30 Cushion: Medium	64151
AirFit F30 Headgear: Standard	64161

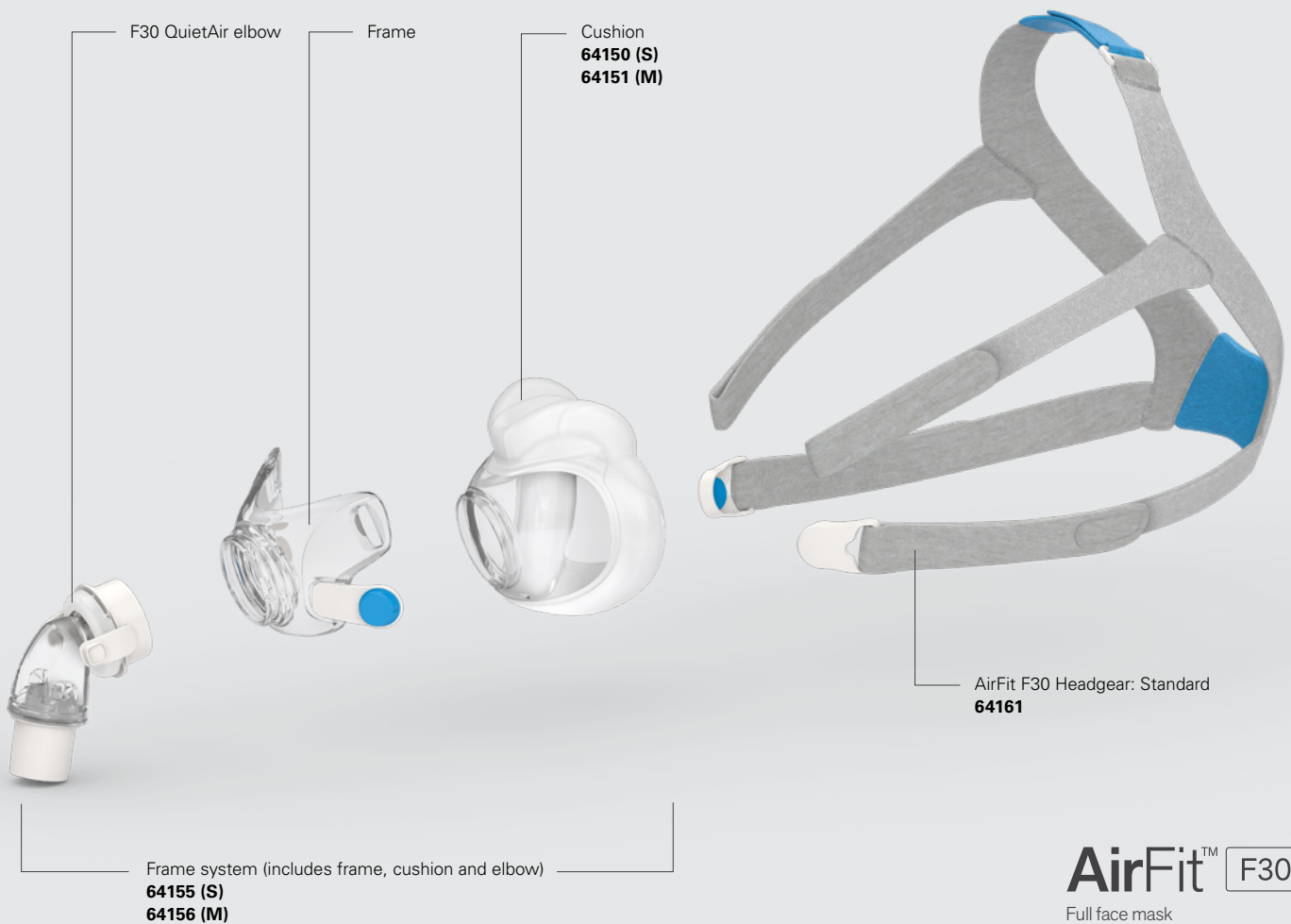



Fitting guide for AirFit F30



- Place the UltraCompact cushion under your nose and ensure it sits comfortably on your face. With the ResMed logo on the headgear facing up, pull the headgear over your head.



- Bring the lower headgear straps under your ears and attach the magnetic headgear clips to the frame.



- Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.



- If the headgear is too close to your ear, adjust the headgear strap on the crown of the head.



- Connect the air tubing from your CPAP machine to the elbow.



- Reattach the elbow to the frame. The mask is now ready to use.



Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

For Sales and Service,

Hitwin Healthcare Pvt. Ltd.

7/10, Jaganathan Nagar 2nd Main Road,
Arumbakkam, Chennai, India - 600106.

Ph: 91 44 4684 5555 (20 Lines), 91 44 4233 3313

Mob: 91 93441 66778

Chat with US

