



Practical innovation

Pico nasal mask delivers value-added performance for you and your patients

Hitwin
HEALTHCARE

Respiratory • Sleep Therapy • Cardiology

Hitwin Healthcare Pvt. Ltd.

7/10, Jaganathan Nagar 2nd Main Road,
Arumbakkam, Chennai, India - 600106.

Ph: 91 44 4684 5555 (20 Lines), 91 44 4233 3313

Mob: 91 93441 66778

PHILIPS

RESPIRONICS

At Philips Respironics, we created Pico with a definitive goal in mind — to create a truly appealing traditional nasal mask that makes minimal contact with the face and offers simplicity, comfort and seal. These qualities make it an ideal choice for new and existing sleep apnea patients who want minimal disruptions to their lifestyle.

Pico is the lightest and smallest traditional nasal mask on the market*

Pico offers a practical alternative that is comfortable, easy to use, and easy to set-up — while providing a high level of therapy. In fact, clinicians rated Pico easier to use than the leading traditional nasal mask, making it ideal for your installed base optimization and new patient set-ups*

Pico comfortably fits more than 98% of patients*

Pico offers patients a simple-to-use mask that maintains its seal through the night, and offers the freedom to do things like watch television with their mask on their face.

*Data on file. Internal Marketing Survey 2013.

Get started

To learn more about how Pico can start making a difference for your patients and your business, contact your Philips Respironics representative for more information.



Item	Part number	HCPCS Code
Small/medium, large, extra large Pico nasal mask Fitpack, with headgear	1104940	
Small/medium Pico nasal mask, with headgear	1104915	A7034,A7035
Large Pico nasal mask, with headgear	1104916	A7034,A7035
Extra large Pico nasal mask, with headgear	1104917	A7034
Small/medium Pico nasal mask, without headgear	1104918	A7034
Large Pico nasal mask, without headgear	1104919	A7034
Extra large Pico nasal mask, without headgear	1104920	A7034
Replacement parts	Part number	HCPCS Code
Pico headgear	1104934	A7035
Pico headgear, reduced size	1104935	A7035
Small/medium nasal cushion	1104936	A7032
Large nasal cushion	1104937	A7032
Extra large nasal cushion	1104938	A7032
Pico sizing gauge	1104939	N/A
Pico headgear clips	1104933	N/A
Pico quick release swivel	1104931	N/A

The ANSWER

Transform your business the easier way

Pico is the latest in our comprehensive portfolio of advanced mask designs plus powerful resupply solutions and innovative technologies. It's all designed to work together to your distinct advantage at a time when you need it most.



Philips Respironics is working in harmony with the care team to produce solutions that can lead to healthier patients, healthier practices and healthier businesses.



Pico fitting guide

Achieving the right fit with Pico is easy; just follow the instructions below to get started.

- 

Before putting on the mask, make sure the cushion fits the width of your nose without blocking your nostrils. The top of the cushion may sit between the bridge and tip of your nose.
- 

Disconnect the headgear clips by twisting them away from the frame like a key and undo the headgear tab by peeling it away from the headgear strap.
- 

Pull the headgear on over your head and gently place the cushion over your nose.
- 

Now, while holding the mask in place, connect the headgear clips by pressing onto the mask frame.
- 

Adjust the top and bottom straps by pulling the tabs back evenly. The mask should rest lightly on your nose. Do not over-tighten. The headgear should fit loose and comfortable.
- 

Next, connect Pico to the flexible tubing that came with your sleep therapy device. Some air leaking is normal. Adjust the side headgear straps to reduce leaks around your eyes.
- 

Assume different sleeping positions. Move around until comfortable. If there are any excessive air leaks, make final adjustments while lying down.